WOONDCLOT

Advanced Bleeding Control™

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Sterile contents

Description:

WoundClot is a water soluble gauze, based on cellulose structure. It is fully biocompatible and provided sterile. In contact with liquids it forms an adhesive gel which expands and adhere to the wound for bleeding control.

WOONDCLOT

Hemostatic Gauze

Advanced Bleeding Control™

INSTRUCTIONS FOR USE (CE)

FOR TEMPORARY CONTROL OF BLEEDING

Sterile contents

Product size:

2.5X2.5cm (1X1"), 5X5cm (2X2"), 5X10cm (2X4"), 10X10cm (4X4").

Indications:

WoundClot gauze is intended for external bleeding control of superficial wounds, minor cuts, and abrasions.

Application:

Place the gauze onto the wound, apply slight pressure and let the blood flow through the gauze until it forms into gel and blood stops flowing. If needed, you may slightly wet the gauze to create a complete gel membrane.

Removal:

Once hemostasis is achieved, remove by saturating the gauze with water or saline solution, then gently wipe any excess gauze material away from the wound.

Cautions:

It is advised to clean the wound with water before using WoundClot. In case of itching, burning, rash or signs of infection such as fever, pain, redness or swelling, seek professional medical care.

WOONDCLOT

Trauma Gauze

Advanced Bleeding Control™

INSTRUCTIONS FOR USE TRAUMA DEVICE (CE)

EMERGENCY HEMOSTATIC DRESSING

For moderate and severe traumatic bleeding

Sterile contents

Product size:

8X20cm (3"X8), 8X100cm (3.1X39").

Indications:

WoundClot Trauma Gauze is intended for bleeding control of traumatic wounds, including moderate to severe bleeding. For use by trained emergency responders.

Application:

1

2

3

4

5

Removal: Once hemostasis is achieved, remove by saturating the gauze with water or saline solution, then gently remove any excess gauze material away from the wound. **Cautions:** It is advised to clean the wound with water before using WoundClot. Bleeding does not always stop immediately. If the bleeding persists pack additional gauze layer and continue to apply gentle constant pressure.

WOONDCLOT

Surgical

Advanced Bleeding Control™

INSTRUCTIONS FOR USE SURGICAL DEVICE (CE)

SURGICAL HEMOSTATIC DRESSING

For surgical procedures

Bio absorbable regenerated cellulose

Sterile contents

Product size:

5X5cm (2X2"), 5X7.5cm (2X3"), 10X10cm (4X4").

Indications:

WoundClot bio-absorbable Surgical Gauze is intended for bleeding control during surgical procedures.

Application:

WoundClot Surgical is dissolvable, keep dry until application. 1. Open sterile pack. 2. Gently place over the wound site. 3. Hold the gauze in the blood flow for a few seconds for adherence, make sure the gauze is soaked with blood. 4. The gel formed, will adhere to the wound and stop the bleeding. 5. If the bleeding persists, apply additional WoundClot Surgical layer.

Removal:

Although WoundClot Surgical is fully absorbable, it is advised to remove excess material once hemostasis is achieved. Saturate injured area with water or saline solution, and gently remove any excess material from the wound.

Storage:

Keep sealed packets in an ambient environment and a dry place for maximum shelf life.

Warnings:

WoundClot Trauma is not intended for wound healing. WoundClot Trauma is not antimicrobial or intended to prevent infections. WoundClot gauze is supplied sterile.

Do not re-sterilize

Do not use after expiration date.

WoundClot gauze is a single use device. Do not reuse.

Prescription only

Consult instruction for Use

Store in dry place

Sterilization using Irradiation

LOT

Batch code

Do not use if package is damaged. Sterility of contents guaranteed unless individual package is damaged or opened.

REF

Catalog number

Manufacturing Date

CORE SCIENTIFIC CREATIONS

Yohanan Hasandlar 4, mail box 2270, Kfar Saba 4464102 Israel
Tel: +972-97881919 | Fax: +972-97790263
info@coreesc.com | www.coreesc.com

EC REP

EC Rep Authorized Representative in the European Community

EU Authorized Representative: CE partner4U BV, Esdoornlaan 13, 3951 DbMaarn, The Netherlands | Fax: +31 343 442 162
CE Marketing and identification number of notified body.

©2014 Core Scientific Creations Ltd, All Rights Reserved. WoundClot is registered trademark of Core Scientific Creations Ltd.

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