|                                                                                                                                                                               |                                                                                                                                                    |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>BAUSCH + LOMB</b></p> <p><b>Dk-line®</b><br/>Perfluorodecalin</p> <p>5ml [REF] VRL100 and 7ml [REF] VRL110</p>                                                          | <p><b>BAUSCH + LOMB</b></p> <p><b>Okta-line™</b><br/>Perfluorocetane</p> <p>5ml [REF] VRL200</p>                                                   |
| <p><b>EN</b> <b>Product Information</b></p>                                                                                                                                   |                                                                                                                                                    |
| <p><b>COMPOSITION</b></p> <p>100% fluorinated perfluorocarbon containing isomers of perfluorodecalin (95–100%), perfluorohydrindane(0–2%) and perfluorocyclobutane (0–4%)</p> | <p>100% fluorinated perfluorocarbon containing isomers of perfluorocetane (&gt;95%), perfluoroleptane and perfluorooctane</p>                      |
| <p><b>CHARACTERISTICS</b></p> <ul style="list-style-type: none"> <li>• Clear fluid of high specific gravity 1.93g/cm<sup>3</sup></li> <li>• Refractive Index= 1.31</li> </ul> | <ul style="list-style-type: none"> <li>• Clear fluid of high specific gravity = 1.77g/cm<sup>3</sup></li> <li>• Refractive Index = 1.27</li> </ul> |

A 100% degree of fluorination results in the complete absence of C-H structures in Dk-line® and Okta-line™ and guarantees a non-toxic product. It is free of preservatives, chemically inert and does not decompose in the eye.

The graph illustrates the spectral transmittance of the contact lens materials. The y-axis represents percentage transmittance (%T) from 0 to 100, and the x-axis represents wavelength in nanometers (nm) from 300 to 1100. The curve is a smooth, slightly downward-sloping arc that remains consistently above 90% transmittance across the entire range, indicating excellent optical clarity.

| Wavelength [nm] | %T    |
|-----------------|-------|
| 300             | ~95   |
| 400             | ~98   |
| 500             | ~99   |
| 600             | ~99.5 |
| 700             | ~99.8 |
| 800             | ~99.5 |
| 900             | ~98.5 |
| 1000            | ~97   |
| 1100            | ~95   |

**INDICATIONS:** Dk-line® and Okta-line® are surgical aids in ophthalmic posterior segment surgery during: retinal detachments/PVR/PDR, giant tears, ocular trauma, and for lifting of subluxated lenses and foreign bodies from the vitreous. **CONTRAINDICATIONS:** Reaction of sub-retinal tissues to liquid perfluorocarbons is not known. Care should be taken to avoid any sub-retinal gravity and low viscosity. Dk-line® or Okta-line® may generate alterations of the retina if present over a long period. Adverse reactions and/or potentially sight threatening complications that may be reasonably regarded as Dk-line® or Okta-line® related and that were not previously expected in nature, severity or degree of incidence should be reported to a Bausch & Lomb representative.

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A 100% degree of fluorination results in the complete absence of C-H structures in Dk-line® and Okta-line® and guarantees a non-toxic product. It is free of preservatives, chemically inert and does not decompose in the eye.

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The graph displays a spectral transmittance curve. The y-axis is labeled 'T%' and ranges from 0 to 100. The x-axis is labeled 'Comprimento de onda [nm]' and ranges from 300 to 1100. The curve is a nearly horizontal line at approximately 95% transmittance, indicating high clarity across the entire visible and near-infrared spectrum.

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