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Immobilisation is a basic principle in the treatment of injuries and overstrain since it greatly accelerates the recovery process and prevents secondary complications. Shoulder-arm supports support damaged tissue structures (joints, ligaments, capsules, cartilage).

The BORT OmoTex is an active support for the treatment of the shoulder joint, has a centering effect on the glenohumeral joint, improves joint tracking and allows limitation of the external rotation of the shoulder joint.

Instruction for use:

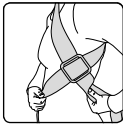
The BORT OmoTex Shoulder Support should be fitted by a health-care professional.

Draw the support over the affected shoulder.

Pull the holding loop in front of and behind the head and put the healthy arm through the holding loop so that the strap runs slanted across the chest, does not constrict the neck and crosses under the opposite shoulder.


Pull the belts with the desired tension through the flexible belt guide plate and pull the abdominal strap behind the upper part of the body.

Draw the vertical abdominal and the back strap through the loop at the level of the umbilicus and optimise the fit of the support by simultaneously pulling on the straps.

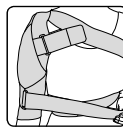The inner silicone pad is in the correct position when it is resting flat on the arm and enclosing the head of the upper arm. If necessary, the tension of the elastic cross straps can be increased in the shoulder region by adjusting the hook loop fasteners.

The enclosed tension strap allows additional limitation of external rotation. To do this, draw the strap through the holding loop on the support and fasten on the abdomen with the hook loop fastener.

Indications:

Chronic, post-traumatic or post-operative irritations of soft tissues in the area of shoulder, i.e. in the case of arthrosis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

Support: 65 % polyamide, 28 % elastodines, 7 % viscose

Pad: 100 % silicone

Important Note:

No reutilization – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

If pressure points, abrasions or similar occur, have your orthopaedic technician or doctor check the fit of the support immediately.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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