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EN

Your physician has prescribed you with a high-quality therapeutic shoe, which is suited for early mobilisation in the form of this aid.

Please observe the following points in order to ensure that wearing this shoe is as comfortable as possible for you.

Instruction for use:

Important: You are not allowed to operate a vehicle when wearing the BORT surgical shoe!

Protect the open foot wound by means of a bandage or similar product. Use crutches when wearing the shoe. Always wear a non-slip shoe with the same heel height on your healthy foot.

Indications:

Post-operative and other wounds, even when treated with extensive and possibly padded bandaging, where the healing process takes place over an extended period of time and weight can be placed partially on the foot whilst walking, even outdoors. An additional insole is not required.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

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