

zur Verfügung gestellt von Gebrauchs.info



Nabelbruchband Neugeborene Navel Hernia Belt for Newborns

Gebrauchsanweisung
user manual

CE

Made in Germany

M E D E S  G N

Dietramszell · Dubai · München

Aktualisierungsstand Gebrauchs.info November 2011

zur Verfügung gestellt von Gebrauchs.info

Nabelbruchband flach hinlegen (langer, weißer Verschuß nach unten). Verband mit nötigen Zug vom Rücken her um den Körper legen. Kleinen losen teil über den mit einer sterilen Gazekompressen bedeckten Nabel legen. Jetzt den Teil mit dem Velcro Verschuß darüber und mit kleinem Verschuß durch Andrücken, schliessen.

54% Baumwolle, 46% Polyamid



Lay down flat bandage (long white fastener downwards). Put bandage with necessary tension form the back around the body. Put the small loose piece over the naval which is covered with a sterilized gauze-compress. Now put the piece with the big velcro-fastener across ans close with the small fastener by pressing against.

54% Cotton, 46% Polyamid



M E D E S  G N

medesign I.C. GmbH · Dietramszellerstr. 6 · D-83623 Dietramszell
Phone +49(0)8027-907 38-0 · Fax +49(0)8027-907 38-27
eMail: info@medesign.de · www.medesign.de